

Method Comparison:

The Ansh IGF-II ELISA has been compared to a Commercial IGF-II assay (Method A) using 28 serum samples in the range of 261 ng/mL to 750 ng/mL. Passing Bablok analysis of the results yielded the following Regression: Ansh IGF-II ELISA (AL-131) = 1.10 (Method A) – 43.31 (r=0.72; P<0.0001).

REFERENCES

1. Daughaday WH, Rotwein P: Insulin like growth factors I and II. Peptide, messenger ribonucleic acid and gene structures, serum and tissue concentrations. *Endocrin Rev* 10:68-91, 1989.
2. Bell GI, Merryweather JP, Sanchez-Pescador R, Stempien MM, Priestley L, Scott J, Rall LB: Sequence of a cDNA clone encoding human preproinsulin-like growth factor II. *Nature* 310:775-777, 1984.
3. Baxter RC, Martin JL, Beniac VA: High molecular weight insulin-like growth factor binding protein complex. *J Biol Chem* 264:11843-11848, 1989.
4. Rechler M: Insulin-like growth factor binding proteins. *Vit Horm* 47:1-114, 1993.
5. Zapf J, Hauri C, Waldvogel M, Froesch ER: Acute metabolic effects and half-lives of intravenously administered insulin-like growth factors I and II in normal and hypophysectomized rats. *J Clin Invest* 77:1768-1775, 1986.
6. Lee PDK, Rosenfeld RG: Clinical utility of insulin-like growth factor assays. *Pediatrician* 14:154-161, 1987.
7. Morgan DO, Edman JC, Standring DN, Fried VA, Smith MC, Roth RA, Rutter WJ: Insulin-like growth factor II receptor as a multifunctional binding protein. *Nature* 329:301-307, 1987.
8. Conover CA, Misra P, Hintz RL, Rosenfeld RG: Effect of an anti-insulin-like growth factor I receptor antibody on insulin-like growth factor II stimulation of DNA synthesis in human fibroblasts. *Biochem Biophys Res Commun* 139:501-508, 1986.
9. Gluckman PD, Ambler GR: What is the function of circulating insulin-like growth factor-2 in postnatal life. *Mol Cell Endocrinol* 97:C1-C3, 1999.
10. Lee PDK, Hintz RL, Rosenfeld RG, Benitz WE: Presence of insulin growth factor receptors and lack of insulin receptors on fetal bovine aortic smooth muscle cells. *In Vitro Cell Dev Biol* 24:921-926, 1988.
11. Kiess W, Greenstein LA, White RM, Lee L, Rechler MM, Nissley SP: Type II insulin-like growth factor receptor is present in rat serum. *Proc Nat Acad Sci USA* 84:7720-7724, 1987.
12. Hintz RL: The somatomedins. *Adv Pediatr* 28:293-317, 1981.
13. Rosenfeld RG, Wilson DM, Lee PDK, Hintz RL: Insulin-like growth factors I and II in evaluation of growth retardation. *J Pediatr* 109:428-433, 1986.
14. Soliman AT, Hassan AEH1, Aref MK, Hintz RL, Rosenfeld RG, Rogol AD: Serum insulin-like growth factors I and II concentrations and growth hormone and insulin responses to arginine infusion in children with protein-calorie malnutrition before and after nutritional rehabilitation. *Pediatr Res* 20:1122-1130, 1986.
15. Powell DR, Rosenfeld RG, Baker BK, Liu F, Hintz RL: Serum somatomedin levels in adults with chronic renal failure: the importance of measuring insulin-like growth factor I (IGF-I) and IGF-II in acid-chromatographed uremic serum. *J Clin Endocrinol Metab* 63:1186-1192, 1986.
16. HHS Publication, 5th ed., 2007. Biosafety in Microbiological and Biomedical Laboratories. Available <http://www.cdc.gov/biosafety/publications/bmbl5/BMBL5>
17. DHHS (NIOSH) Publication No. 78-127, August 1976. Current Intelligence Bulletin 13 - Explosive Azide Hazard. Available <http://www.cdc.gov/niosh>.
18. Kricka L. Interferences in immunoassays – still a threat. *Clin Chem* 2000; 46: 1037–1038.

FOR RESEARCH USE ONLY

Not for use in diagnostic procedures.

The Ansh Labs logo is a trademark of Ansh Labs.



Manufactured by:

Ansh Labs

445 Medical Center Blvd.

Webster, TX 77598-4217, U.S.A.

For Illustrative Purposes Only
 Refer to package insert included with
 the product for exact specifications.